


# TNT Medpak Frozen 5 Packing Instructions



[1] Remove any outer packaging until you reach the TNT printed outer carton. Open this carton and remove the insulating box lid.



[2] Prepare the samples by placing them in the absorbents, and putting inside the pouch. Seal by following the instructions printed on the pouch.

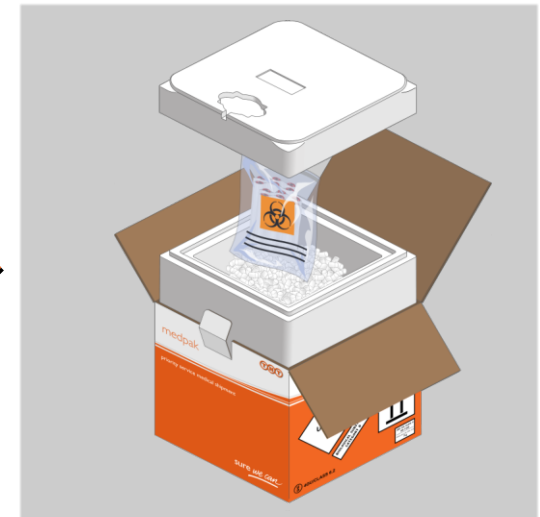

[3] Place the sealed pouch on top of the dry ice fill inside the insulating box. Apply the insulating box lid.



[6] Affix completed shipping documentation to the carton lid.



[5] Close the carton and secure tabs by applying security seal.



[4] Complete itemised list of contents, and place inside the outer carton.